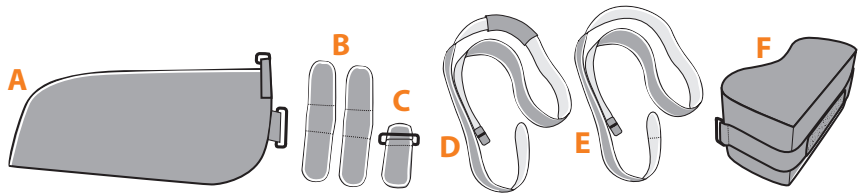
INSTRUCTIONS FOR USE

20116 ABDUSLING™ - Shoulder abduction 15°

Package included

- A. Mitella / Mitella / Fatle / Sling / Mitella
- B. Fäste x2 / Lukkemekanisme x2 / Feste x2 / Fastening x 2 / Kiinnikkeet, 2 kpl
- C. Fäste med vändsölja / Lukkemekanisme med D-ring / Feste med spenne / Fastening with slide buckle / Kiinnike ja lenkki
- D. Axelband med polstring/ Skulderstrap med polstring / Skulderstropp med polstring / Padded shoulder strap / Pehmustettu olkahihna
- E. Midjeband / Midjebånd / Midjebånd / Waist strap / Vyötäröhihna
- F. Abduktionskudde med kardborre fäste / Abduktionspude med velcrolukning / Abduksjonspude med borrelåsfeste / Abduction pillow with Velcro fastening / Abduktiotyyny, jossa tarrakiinnitys

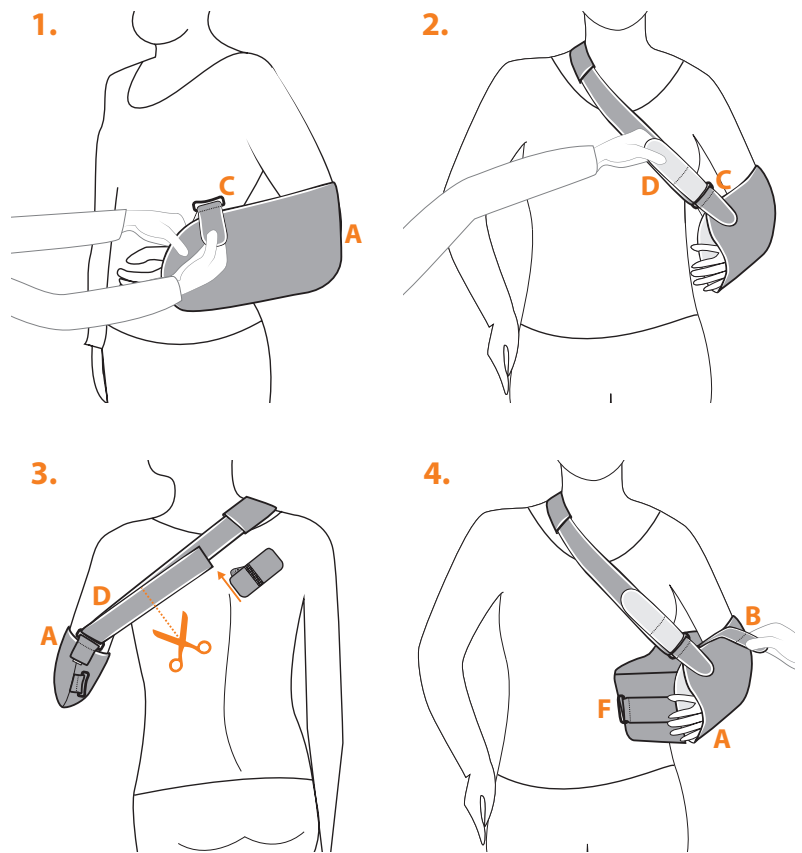
Instructions for use

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- EN** 1. Position the arm in **A** so that the elbow rests against the rear edge. Close with **C** on top.
2. Pull **D** through the slide buckle on **C**. Secure **D** approx. 15 cm above.
3. Pull **D** through the slide buckle on **A**. Open the crocodile clip. Cut **D** to the desired length. Refasten the crocodile clip to the strap and secure in place.
4. Position **F** against the hip at an appropriate height, with the broader part facing forward. Secure **A** to the Velcro fastening on the pillow. Secure **B** on top of **A** to fix the arm in the position required.

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Red: 2017-05-09

This product meets the requirements of
the Medical Devices Directive 93/42/EEC.
Class 1 medical device, non sterile.

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9. Tilpasset produkt forfra.

10. Tilpasset produkt bakfra.

EN 5. Draw **E** through the front slide buckle on the pillow. Secure **E** approx. 15-20 cm along the strap.

6. Draw **E** through the rear slide buckles on the pillow, first through the buckle on **F** and then through the buckle on **A**.

7. Open the crocodile clip. Cut **E** to the desired length. Refasten the crocodile clip to the strap and secure in place.

8. Secure **B** forwards over **A** as support for the thumb, if required.

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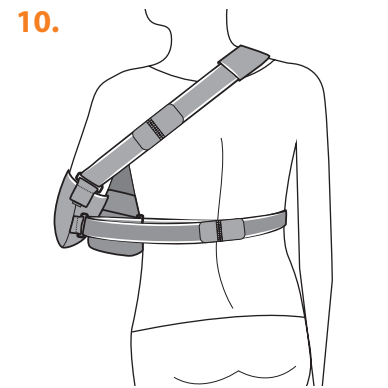
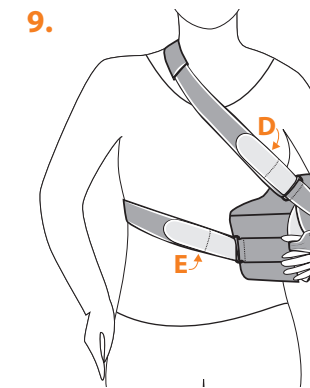
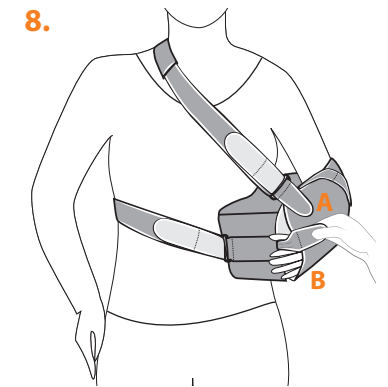
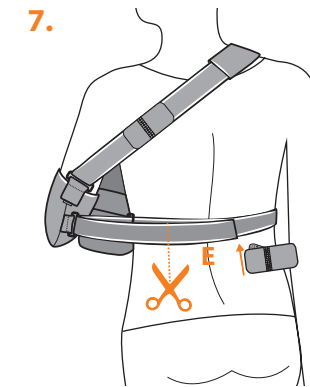
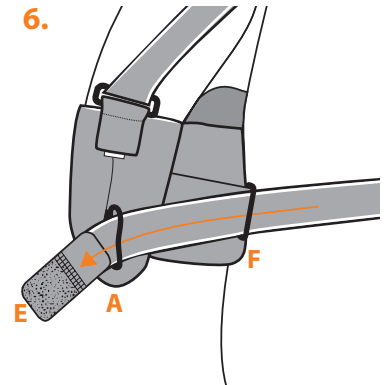
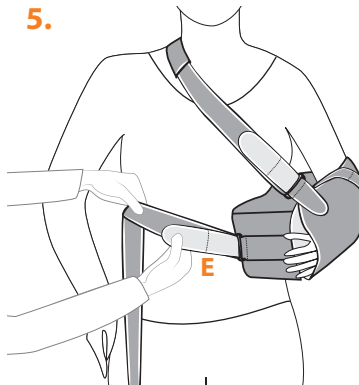
EN Open fastenings **B+B**, loosen shoulder strap **D** and waist strap **E** at the front.

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CE This product meets the requirements of the Medical Devices Directive 93/42/EEC. Class 1 medical device, non sterile.

Manufactured by:
PRIM, S.A.
c/F, no 15,
Pol. Ind. no 1
28.938 Móstoles
Madrid, Spain

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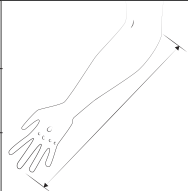
EN To relieve the shoulder/arm when healing from acute injury/trauma and for post-operative treatment of shoulder injury.

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Measurement

Measure from elbow to fingertips.
Fits both left and right arm.

Size	cm	
S/M	35-44	
L/XL	43-55	

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EN Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage.

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Composition

Textiles materials except pillow: 2% polyamide, 26% polyurethane, 2% cotton, 70% polyester.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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EN Wedge pillow for increased abduction to more than 15°, art.no. 20117, to be ordered separately.

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Red: 2017-05-09

 This product meets the requirements of
the Medical Devices Directive 93/42/EEC.
Class 1 medical device, non sterile.

 **Manufactured by:**
PRIM, S.A.
c/F, no 15,
Pol. Ind. no 1
28.938 Móstoles
Madrid, Spain