INSTRUCTIONS FOR VICTORIA – POST SURGICAL BRA

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EN The bra can be opened both in front and shoulder straps, allowing easy application and adjustment after surgery and/ or during recovery.

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EN Worn after breast surgery. The supportive bra keeps implants in position and reduces the risk of swolleness and bleeding after surgery, as well as enables a fast healing process and provides pain relief.

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EN 1. Tight circumference measurement around bust = X. Round up or down to the nearest multiple of 5 (eg measure 78 cm gives a bra size of 80)


2. Loose measurement over the fullest part of the breast, from the centre of the spine to the centre of the sternum = Y. Multiply this measurement by two = Y2.

3. The difference between Y2 and X decides the cup size. (Y2 - X = cup size)

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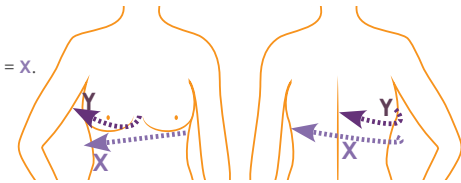EN In order to determine which size to wear, please fit the bra. The bra is to fit tightly, but not feel uncomfortable. Compression can be regulated by choice of size.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of compression garments. For best results, follow instructions for use.

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Materials

Polyamide 70%, Elastane 15%, Cotton 15%. Latex free.

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EN Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). Washing in lower temperatures will increase the longevity of the bra. Flat or line dry. Do not tumble dry. Close hook and eye before washing. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.