INSTRUCTIONS FOR SANDRA – POST SURGICAL BRA

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EN Stable, seamless and super soft bra. Wide, adjustable shoulder straps, back closure. Recommended for use directly after breast surgery. Also suitable as a stage-two bra or as a prosthesis bra as it has pockets for padding/breast forms.

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EN Use the size guide and size schedule to decide the right size. The fit should be tight but comfortable. The degree of compression can be adjusted by choosing a larger/smaller size.

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EN 1. Tight circumference measurement around bust = X. Round up or down to the nearest multiple of 5 (eg measure 78 cm gives a bra size of 80)

2. Loose measurement over the fullest part of the breast, from the centre of the spine to the centre of the sternum = Y. Multiply this measurement by two = Y2.

3. The difference between Y2 and X decides the cup size. (Y2 - X = cup size)

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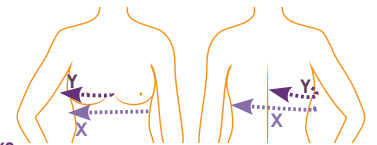
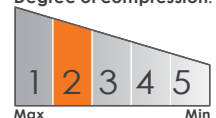
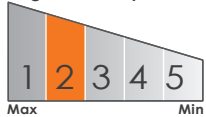
EN If you experience pinching, chafing, or color changes in the skin, remove the bra and contact the product fitter. If you recently went through breast surgery, fitting, prescribing and recommendations of use, should be done by or under the supervision of a medical professional.

Material composition

Polyamide 79%, Elastane 11%, Polyester 10%. Latex free.

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EN Machine wash at maximum 60°C (140 F), for longer durability, 40°C (104 F) is recommended. Fasten hook- and- eye closure before washing. Use of a laundry bag is recommended. Possible to tumble dry at low heat. Do not iron. If the garment has any blood stains, soak or wash in cold water. Do not use bleach, chlorine or fabric softener as this could cause the material to lose functionality.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.