
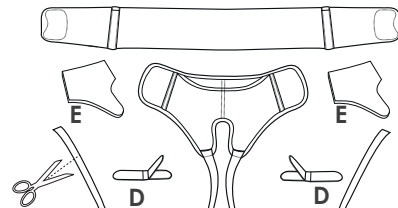
# INSTRUCTIONS FOR USE



## 50215, 50216, Balder - HERNIA BELT

### 50215 - Balder Bilateral:

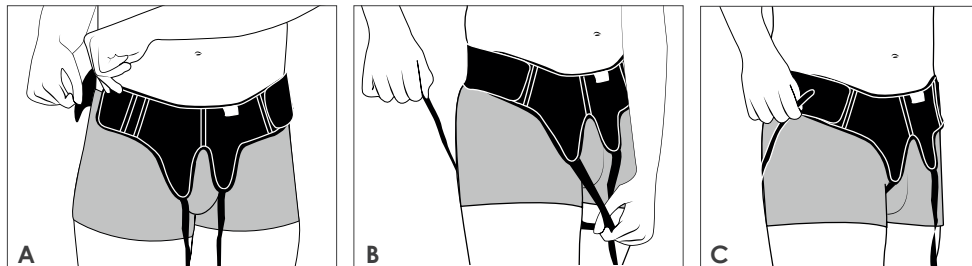50215 Balder Bilateral: D+E x 2 is included

### 50216 - Balder Unilateral: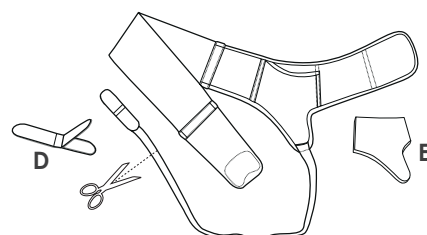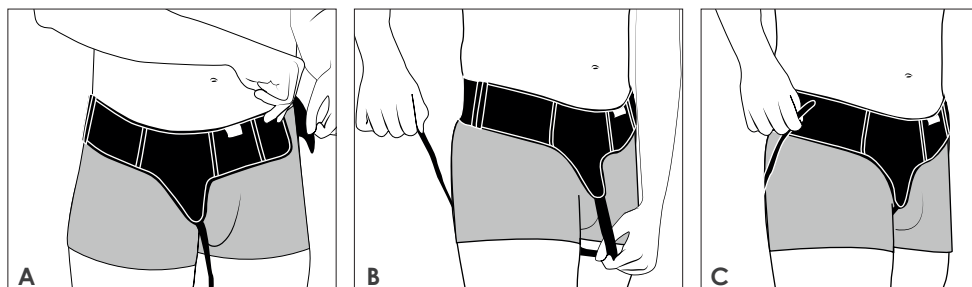

50216 Balder Unilateral: D+E x 1 is included

### Application

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**EN** 1. Fit the hip belt **A** into place. Adjust for steadiness and comfort.  
2. Take the long strap/straps **B** and pull it back between legs. Pull up over the hip and attach **C**. If the straps are too long, open the Velcro attachment **D** and cut the strap to desired length.  
3. Adjust for comfort and sufficient of pressure. Included pads **E** should be placed in the pockets. Using boxer shorts with the belt increases comfort and minimizes the risk of soreness.

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**EN** Inguinal hernia. Pre- or post op, or with conservative treatment. **Counter indication:** incarcerated inguinal hernia.

### Measurement

Size	Hip Circumference
X Small	70 - 85 cm
Small	80 - 95 cm
Medium	90 - 105 cm
Large	100 - 115 cm
X-Large	110 - 125 cm

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**EN** In order to determine the correct size, measure the hip circumference (at its widest point).

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Check that the Velcro is free from dirt and that the fastening function is working. Consult medically trained personnel if using this product during medication and/or in combination with aids of assistance.

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**EN** Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

### Material

**50215 Balder Bilateral:** 56% Polyamide, 23% Polyester, 15% Polyurethane, 6% Rubber. Pads: Polyethylene foam.

**50216 Balder Unilateral:** 56% Polyamide, 23% Polyester, 15% Polyurethane, 6% Rubber. Pads: Polyethylene foam.

Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner or medical equivalent - or Nordicare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.