INSTRUCTIONS FOR VILMER – COMPRESSION VEST

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EN 1. Make sure the skin is dry. Open the zipper and hook-and-eye.
2. Slip into the vest. - First, close all hook-and-eyes on the side. Then close zipper.
3. Adjust the shoulder straps if necessary.

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EN In order to determine which size to wear, please fit the garment. The compression garment is to be fitted tightly, but not feel uncomfortable. After some time, a smaller size can be used.

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EN Worn after plastic surgery to ensure best possible result. Compression significantly reduces the risk of swollenness and bleeding after surgery, as well as enables a fast healing process and provides pain relief. Vilmer is recommended after breast reduction/ gynecomastia and liposuction.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of compression garments. For best results, follow instructions for use.

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EN Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). Washing in lower temperatures will increase the products longevity. Flat or line dry. Do not tumble dry. Close zipper before washing. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

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Please consult our product catalogue for more information.
For further questions, contact your doctor, health practitioner,
medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907,
info@nordicare.se.