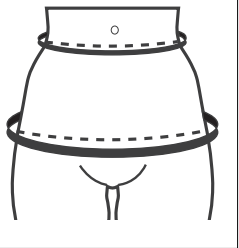
INSTRUCTIONS FOR PAULINA – BODY WITH LONG LEGS

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- EN** 1. Make sure your skin is dry. Open the zippers and unhook the hooks.
2. Carefully pull the bodysuit up over knees/thighs. Make sure the crotch is positioned right, and then pull the bodysuit up the rest of the way and put on the shoulder straps.
3. Fasten the eyes/hooks on the side and then fasten the zipper.
Adjusting leg length: Cut below the marked seam (see the figure above).

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Hofte: Stå med føttene samlet og mål omkretsen der hofte er på det bredeste.

EN Use the size chart to find your size. The garment should fit tight without feeling uncomfortable. Sometime after surgery, once the swelling has come down, a smaller size may be necessary.

NOTE: Compression garments are designed to fit tight. Careful measuring is required to achieve optimal compression and support.

Waist: Measure the circumference at the narrowest point of the waist.

Hips: Stand with both feet together and measure the circumference at the widest point.

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EN The Paulina bodysuit is recommended after a tummy tuck and/or after liposuction on the abdomen, back, buttocks, thighs, knees and calves and/or after plastic surgery on the thighs. Compression improves circulation and reduces the risk of post-operative swelling, leading to better healing (and even) as well as pain relief.

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the bodysuit and contact a professional to help you out testing the product. Compression garment testing, prescribing, and recommendation of use should be handled by or under the supervision of a healthcare professional. For best results, always follow the user instruction.

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EN Machine wash at 40°C. Can be washed at a higher temperature if necessary, max. 60°C. The product will last longer if washed at a lower temperature. Use a laundry bag. Do not tumble dry. Close the zipper before washing. If there are blood stains on the garment, soak or wash it in cold water. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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Materials

85% Polyamide 15% Elasthane. Latex free.

Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.