INSTRUCTIONS FOR KIM – POST SURGICAL FACIAL BANDAGE

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EN Possible to trim and cut the material and straps for optimal fit. For example, trim around the cheeks, chin and neck, or cut holes for the ears.

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EN At facelift and/or liposuction of the chin/neck, ear surgery or when the steady compression under the chin, around the neck, cheeks, hairline or ears is desired.

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Hals: Det bredeste området midt på halsen.

Hode: Det bredeste området over pannen.

EN Use the size guide to decide the correct size. The sizes are flexible and can be adjusted and trimmed for desired compression and optimal fit.

Neck: The widest point in the middle of the neck.

Head: Widest point over the forehead.

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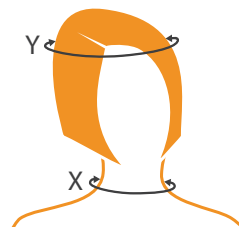
EN Machine wash at 40°C (104°F), can if necessary be washed at higher temperatures, up to 60°C (140°F). The product will last longer if washed at a lower temperature. Dry flat. Do not tumble dry. Close the Velcro fastening before washing, use a washing net. In case of blood stains, soak or wash in cold water. Do not use bleach or fabric softener as this can impair the function of the material.

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Materials

Polyamide 76%, Elastane 14%, Polyurethane 10%. Latex free.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of compression garments. For best results, follow instructions for use.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.