# DuoPlus

art no 20113

## SHOULDER IMMOBILIZER

This packaging can be recycled as paper and the bag as plastic.

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**EN** Two-piece shoulder immobilizer that immobilize and fixates the shoulder and upper arm without causing pressure on the neck. The two shoulder straps crosses in front over the back and distributes the weight over each shoulder. To be used in case of acute injuries, trauma or postoperative. Four sizes, universal left/right.

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**EN** To facilitate application of the shoulder immobilizer, ask someone to assist you. Put the shoulder immobilizer on in front of a mirror to ensure fit is correct.

1. Place the arm in the arm pocket and place one shoulder strap over each shoulder.
2. Cross the straps in the back and bring the straps under the arm pocket. Secure the straps Velcro on the front of the pocket.
3. The immobilisation band is to be attached to the arm pocket, positioned just above the arm. The band is pulled around the chest, back and upper arm and fastened at the front of the pocket. Ensure that the hand and fingers can move freely.

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**EN** For immobilization / fixation of shoulder / arm. In case of trauma, emergency or postoperative care.

**Contraindications:** For patients with sinus carotid sensitivity, avoid pressure from the orthosis in the carotid sinus area.

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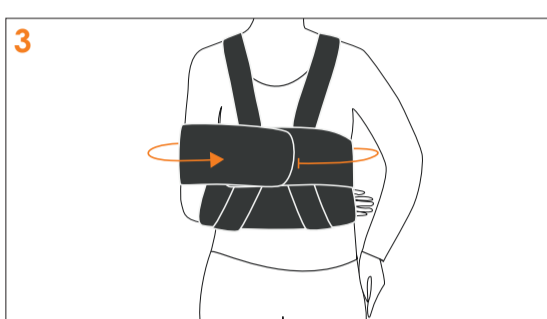
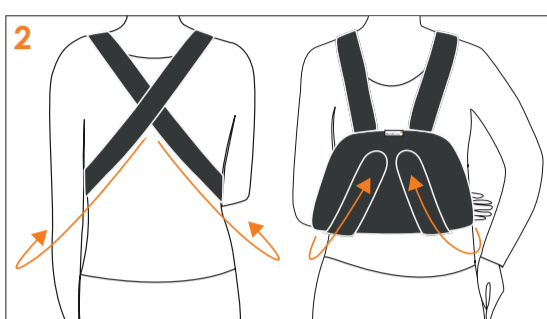
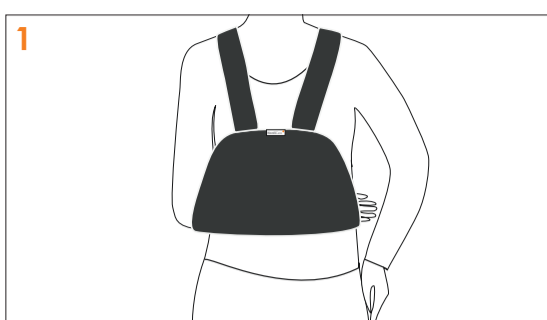
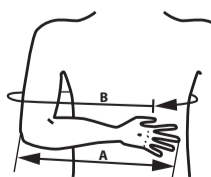
**EN** **A) Length forearm:** Elbow to finger tips in cm.

**B) Torso + upper arm:** Measure around the intended placement over the torso and one arm.

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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**EN** Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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## Material Composition

Nylon 40%, Polyester 25%, PU Foam 20%, Cotton 15%. Latex free.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se). For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se).