MammaMia

art no 50102

MATERNITY BELT

EXTRA SUPPORT & LIFT

2022-09-28

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EN 1. Fully loosen the outer adjustable straps. Position the girdle and secure the broad Velcro closure under the belly.

2. Adjust to fit the shape of the belly so that it feels comfortable.

3. Tighten the adjustable straps until desired support; then fasten the straps below the belly. Make sure not to tighten too hard.

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EN **Indications:** Pregnancy-related pelvic girdle pain (PGP) or symphysis pubis dysfunction (SPD). Also suitable for non-pregnant people seeking stable support for the pelvic joints.

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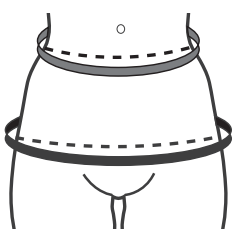
Material

Ventilated, breathable textile material that is gentle on the skin. Polyester 51%, Polyamide 17%, Rubber 12%, Polyurethane 8%, Cotton 7%, Elastane 5%.

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EN Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the velcro to ensure they are free from dirt and damage. In case of pain, chafing or colour changes in the skin, remove the orthosis and contact the fitter. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash, 40°. Close the Velcro fastenings. Use a washing net. Do not bleach, do not dry clean. Do not iron or tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.