MammaMia

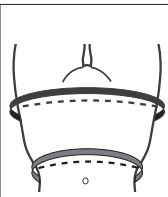
art no 50102

MATERNITY BELT

EXTRA SUPPORT & LIFT

Size	Hip circumf. in cm
2XL	120-135
XL	110-125
L	100-115
M	90-105
S	80-95

SE-263 62 Viken
Sollösingen 1,
Sweden
Phone +46 42 35 22 20,
Fax +46 42 35 22 21

NordiCare®
info@nordicare.se
www.nordicare.se

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EN 1. Fully loosen the outer adjustable straps. Position the girdle and secure the broad Velcro closure under the belly.

2. Adjust to fit the shape of the belly so that it feels comfortable.

3. Tighten the adjustable straps until desired support; then fasten the straps below the belly. Make sure not to tighten too hard.

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EN **Indications:** Symphysiolysis during or after pregnancy. Pelvic pain. **Contraindications:** None known.

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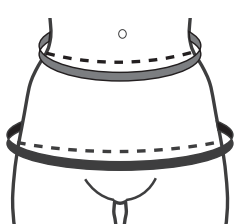
EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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Material

79% Polyester, 7% Spandex, 8% Nylon, 6% Natural Rubber*

*Latex is covered does not come in contact with skin.

Washing Instructions

Machine wash, 40°C. Close the Velcro fastenings. Use a washing net. Do not bleach, do not dry clean. Do not iron or tumble dry.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.