INSTRUCTIONS FOR USE

Lyft – ABDOMINAL BINDER

art no 50206

Application

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- EN** 1. Put the binder in place. Close Velcro on the side.
2. Adjust until comfortable. Make sure it is not too tight.

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EN Machine wash at 40°C (104°F). Can be washed up to 90°C (194°F) (high temperatures will reduce the life time of the Velcro). Seal the Velcros. Use washing bag. Do not bleach. Do not dryclean. Do not iron. Do not Tumble Dry.

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EN Lyft is an elastic abdominal girdle with an abdominal shape to provide support and lift. Provides convenient and efficient support and stability for people with hefty/large abdominal and/or hernia problems. Also used in postoperative care after abdominal surgery, relieves ulcers, prevent incisional hernia and enables early mobilization.

Heights and Measurement

Lyft is 24 cm high at front and 23 cm high at the back. For correct size measure the hip circumference in cm.

Size	Hip Circumference
Small	80 - 95 cm
Medium	90 - 105 cm
Large	100 - 115 cm
X Large	110 - 125 cm
2X Large	120 - 135 cm

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EN Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage.

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EN Elastic abdominal binder with an abdominal shape which provides a comfortable and efficient support/ compression of the abdomen.