## Balder

art no 50215

**HERNIA BELT**

Size	Hip Circumference
XL	110 - 125 cm
L	100 - 115 cm
M	90 - 105 cm
S	80 - 95 cm
XS	70 - 85 cm

**NordiCare®**  
Solfsvägen 1,  
SE-263 62 Viken  
Sweden  
Phone +46 42 35 22 20,  
Fax +46 42 35 22 21  
info@nordicare.se,  
www.nordicare.se

2023-09-21

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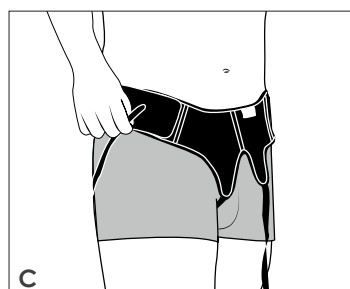
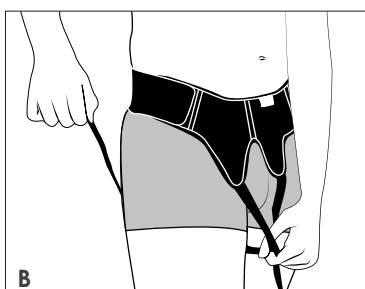
**EN** Balder is a discreet inguinal hernia support belt, well covered underneath clothing. The belt provides good support and is suitable for smaller to medium hernias. Available in bilateral and unilateral design.

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**EN** 1. Fit the hip belt **A** into place. Adjust for steadiness and comfort.  
2. Take the long straps **B** and pull them back between legs. Pull up over the hip and attach **C**. If the straps are too long, open the Velcro attachments **D** and cut the straps to desired length.  
3. Adjust for comfort and sufficient of pressure. Included pads **E** should be placed in the pockets. Using boxer shorts with the belt increases comfort and minimizes the risk of soreness.

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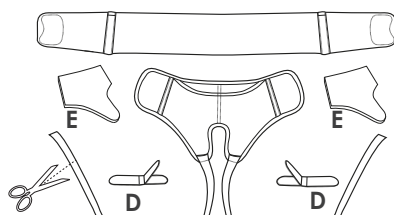
**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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50215 Balder Bilateral: Part D+E x 2 is included.

### Material

80% Polyamide, 14% Polyester, 6% Natural rubber latex\*.  
Pad: EVA foam.  
\*Latex is covered does not come in contact with skin.

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**EN** Inguinal hernia. Pre- or post op, or with conservative treatment. **Contraindications:** Incarcerated inguinal hernia.

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**EN** In order to determine the correct size, measure the hip circumference at its widest point.

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**EN** Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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