StomiSoft

art no 50218, 50219

STOMA BINDER

StomiSoft – STOMA BINDER

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info@nordicare.se,
www.nordicare.se

MD Class I

This packaging can be recycled as paper and the bag as plastic.

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EN Straight elastic stoma binder to be used as stomy support/hernia belt with or without hole for the stoma. Suitable as permanent binder. The binder has a non-elastic front which is possible to cut. Easy to seal and adjust with functional handle and velcro fastening in the front. Reversible to suit stoma on both right and left side. Available in two heights.

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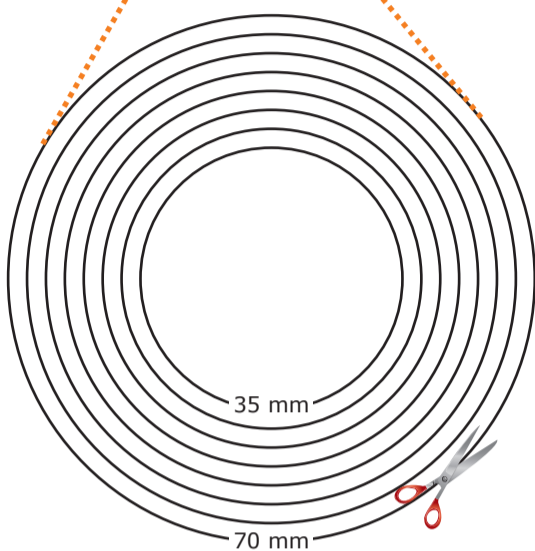
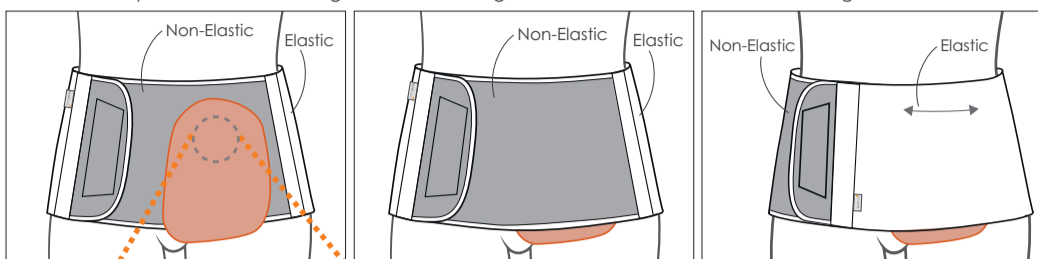
art no 50218, 50219

3 possible applications

1. With cut hole for stoma bag for strong compression around stoma but no compression on stoma bag.

2. Stoma bag under non-elastic front for stronger compression on stoma bag.

3. Stoma bag under elastic material for less compression on stoma bag.

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EN ENFCOS fastens on the entire surface of the non-elastic front material and makes the binder flexible in size and circumference, the binder can be adjusted to attain desired pressure. The binder should not be too tight. Reversible to suit stoma on both right and left side. Suitable to use with or without cut hole.

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EN Stabilizing girdle for hernia/stoma/ostomy hernia. Prevents herniation. Fixation of stoma bandage. Mobilization after surgery. **Contraindication:** incarcerated hernia.

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EN Measure circumference around intended placement. Pick the correct size according to the measurement table.

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EN If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioners. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash at 40°C (106°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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Material

Polyester 49%, Polyamide 29%, Elastane 11%, Polyurethane 11%. Latex free.

For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, info@nordicare.se.