
NordiCare<sup>®</sup> +

**Mika**

art no 50224




**FIXATION BELT**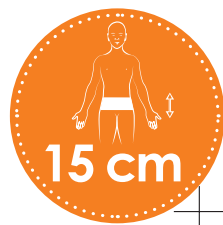

# INSTRUCTIONS FOR USE

## Mika - FIXATION BELT

art no 50224

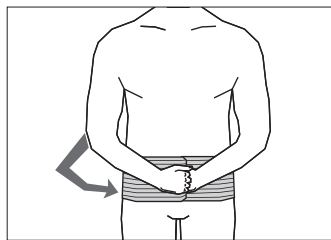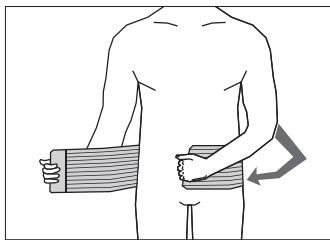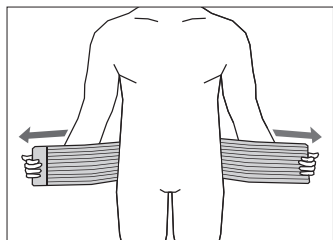### Application

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**EN** If a stinging sensation, rash or other changes in skin color occur, discontinue use and contact your doctor or other medically trained person. Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Periodically check the condition of the Velcro to ensure they are free from grease and damage. Use in combination with other medical products should be supervised by qualified medical practitioner. Serious incidents shall be reported to NordiCare and the competent authority.

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**EN** Soft fixation band for fixation of dressings or to be used as a breast band. The fixation band is discreet and invisible under clothing. Mika comes in two flexible sizes, easy to adjust as the Velcro attaches to the surface of the entire band. Mika is available in white. Height 15 cm.

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**EN** Machine wash at 60°C (140°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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This packaging can be recycled as paper and the bag as plastic.