Vidar

art no 50251

JOCKSTRAP

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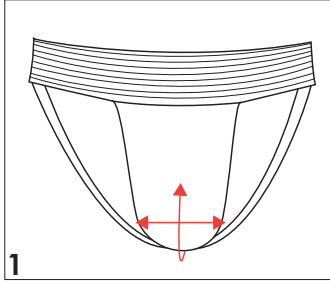
EN Discreet jockstrap with high comfort. Provides maximum support and protection in the scrotal area, without uncomfortable pressure of the abdomen. Seamless design for optimum comfort.

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EN Discreet jockstrap with maximum support. Provides lift and support as needed (1).

- To be worn directly towards the skin. Do not use the jockstrap directly on damaged/irritated skin.
- Do not use with lotions or creams.

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
EN Scrotal hernia. Pre-/postoperatively or conservative treatment. To protect/keep in place after surgery/injury or during sports activity.

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Materials

Cotton 41%, Polyamide 25%, Polyester 13%, Elastane 12%, Lycra 9%.

Size	A
X Small	70-80
Small	81-90
Medium	91-102
Large	103-113

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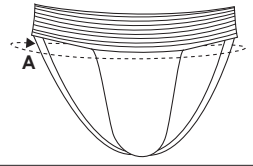EN Prescription, initial fitting and treatment recommendations are to be made by a qualified medical practitioner. A qualified medical practitioner should also supervise when using the product in combination with other medical products. Follow instruction for use. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Machine wash at 40°C (104°F). Use washing bag. Do not bleach. Do not dry clean. Do not tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, medical practitioner or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.