# Loke

art no 50252

HERNIA BRIEFS

2023-09-11

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**EN** To be worn instead of regular briefs. Flat pads (1) are included, convex pads (2) for increased pressure can be ordered separately. The trousers provides lift and support (3).

- Place the pads in the pockets. Convex pads should be positioned with the convex part against the body. Remove pads before washing briefs.
- Put on the briefs first thing in the morning. Preferably laying down, when the hernia is most sunken.
- The fly is easily opened with velcro closure, when going to the toilet.
- To be worn directly against the skin. Do not use the briefs directly on damaged/irritated skin.
- Do not use with lotions or creams.

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**EN** For conservative treatment of light/medium inguinal hernia also postoperatively to support the area of surgery and provide pain relief. **Contraindication:** Incarcerated hernia.

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**EN** Only medically trained persons are to prescribe and recommend the use of the product. For best results, follow instructions for use. Use in combination with other medical products should be supervised by qualified medical practitioners. Serious incidents shall be reported to NordiCare and the competent authority.

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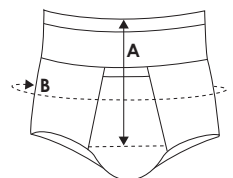
**EN** Hip circumference (B) in cm. Height is measured from the pubis and up (A).

Size	A	B
02	18	76-80
03	18	81-85
04	18	86-90
05	18	91-95
06	18	96-100
07	18	101-105
08	18	106-110
09	18	111-115
10	18	116-120

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**EN** Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not tumble dry.

### Materials

Polyamide 54%, Cotton 32%, Rubber\* 14%. Pads in Polyethylene. \*Rubber is covered, does not come in contact with skin.

For more information please consult our product catalogue or visit [www.nordicare.se](http://www.nordicare.se). For further questions, contact your doctor, health practitioner or medical equivalent - or NordiCare: +46 42-35 22 20, [info@nordicare.se](mailto:info@nordicare.se)

NordiCare®

2023-09-11

Solrosvägen 1, SE-263 62 Viken, Sweden  
Phone +46 42 35 22 20, Fax +46 42 35 22 21  
[info@nordicare.se](mailto:info@nordicare.se), [www.nordicare.se](http://www.nordicare.se)

Made by: SAFTE S.p.A  
Via Adamello 13  
10093 Collegno (TO), Italy