INSTRUCTIONS FOR USE

Dolly – HIGH BACK BODY

art.no 51207

Application

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EN 1. Make sure your skin is dry. Open the hook and eye closure.

2. Carefully pull the body up to the hip. Make sure that the crotch part is high/comfortable in the crotch and that the body is centered at the back.

3. Then pull the body over your stomach and back, make sure to pull your back properly so that it gets high before you tighten the shoulder straps.

4. Close the hook/eyes at the front to the desired size/shape/compression. Make sure the flap behind the hook/eyes is flat against the skin.

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EN Recommended after surgery on the buttocks, hips, abdomen, and back. Compression improves circulation and reduces the risk of post-operative swelling, leading to better healing and even pain relief.

Contraindications: Inflammation in the surgical area. Necrosis.

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EN If you experience a stinging sensation, chafing or color changes in your skin, remove the garment and contact the professional helping you to test out the product. Fitting, and recommended use should be handled by or under the supervision of a medical professional. For best results, follow the user instructions. Serious incidents shall be reported to NordiCare and the competent authority.

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EN For best fit and correct compression it is important that measurements are taken carefully and size chosen according to the measuring table. The fit of the garment should be tight but comfortable. If the patients measurements differ from the measurement table, the garment should not be used. Keep in mind that measurements may vary during the healing process, a time after surgery when any swelling has gone down, a smaller size may be required.

Waist: Measure the circumference at the narrowest point of the waist.

Hips: Stand with both feet together and measure the circumference over pubis, where hip/seat is at its widest.

Thigh: Circumference measurements at the middle of the thigh.

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EN Machine wash at 40° C. Can be washed at a higher temperature if necessary, max. 60° C. The product will last longer if washed at a lower temperature. Use a laundry bag. Do not tumble dry. If there are blood stains on the garment, soak or wash it in cold water. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner or medical equivalent or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.