Knee-O-Trakker w hinge

art.no 70611

PATELLA BRACE

Measurement

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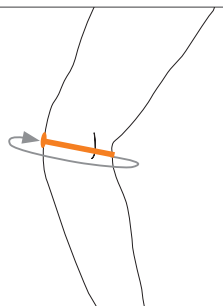
EN Circumference around knee, over patella.

	Size	Circumference cm
Child	XS	30,5-33
	S	33-35,5
	M	35,5-38
	L	38-40,5
	XL	40,5-46
King-size	2XL	46-51
	3XL	51-56
	4XL	56-61

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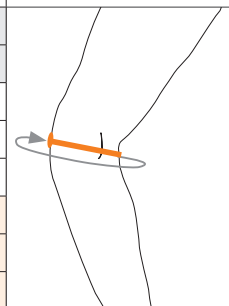
- EN** 1. Loosen straps, making sure they remain threaded through D-rings.
2. Loosen and position the C-shaped pad on affected side of the patella (kneecap) and slide sleeve over knee.
3. Make sure seam is placed in back of leg.
4. Kneecap (patella) is centered in the opening with pad placed closed to edge of patella.
5. Tighten and fasten the narrow straps, thereafter the wider ones. Adjust straps to desired tension or comfort level.

Material Composition

Latexfree Neoprene 70%, EVA foam 10%, Aluminum 5%, Elastic 15% (20% Elastane, 80% Nylon)

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EN Subluxation/luxation of patella and patellofemoral pain syndrome (PFPS). Stabilizing joints in aluminum which are easy to bend for optimal fit.

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EN Serious incidents shall be reported to NordiCare and the competent authority.

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EN Hand wash max 30° C. Dry flat. Seal the Velcros and remove the hinges before washing. Do not use bleach, chlorine bleach or fabric softener as this could impair the function of the material. Do not dry clean. Do not iron. Do not tumble dry.

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