INFORMATION FOR

Cirrus Heel Wedge - FOR WALKERS

art no 80215

Indications

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EN Accessories for Cirrus Airwalker, wedges for increased plantar flexion.

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EN Always read all instructions carefully before using this product. When in doubt, consult your doctor/device fitter. It is important that you follow the general and individual user instructions you receive from your doctor/device fitter throughout your rehabilitation period. Prescription, initial fitting and recommendation on how to use this product should be made by medically trained person. If changes in color of skin or other blood circulation related problems and skin irritation occurs, immediately remove the brace. The brace is only for one patient use. Serious incidents shall be reported to NordiCare and the competent authority.

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EN Clean with a soft damp cloth use mild detergent.

Material Composition

100% EVA-foam.

NordiCare®

Measurements & Size

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EN Universal, left/right.

NO Universal, venstre/høyre.

Cirrus, Left heel wedge	80215-10-00
Cirrus, Right heel wedge	80215-20-00

Cirrus Heel Wedge

art no 80215

HEEL WEDGES FOR WALKERS

For more information, see productleaflet or www.nordicare.se. You can also contact your fitter or NordiCare: +46 42-35 22 20 / DK 7070-1907, info@nordicare.se.

MD Class I **CE**

This packaging can be recycled as paper and the bag as plastic.

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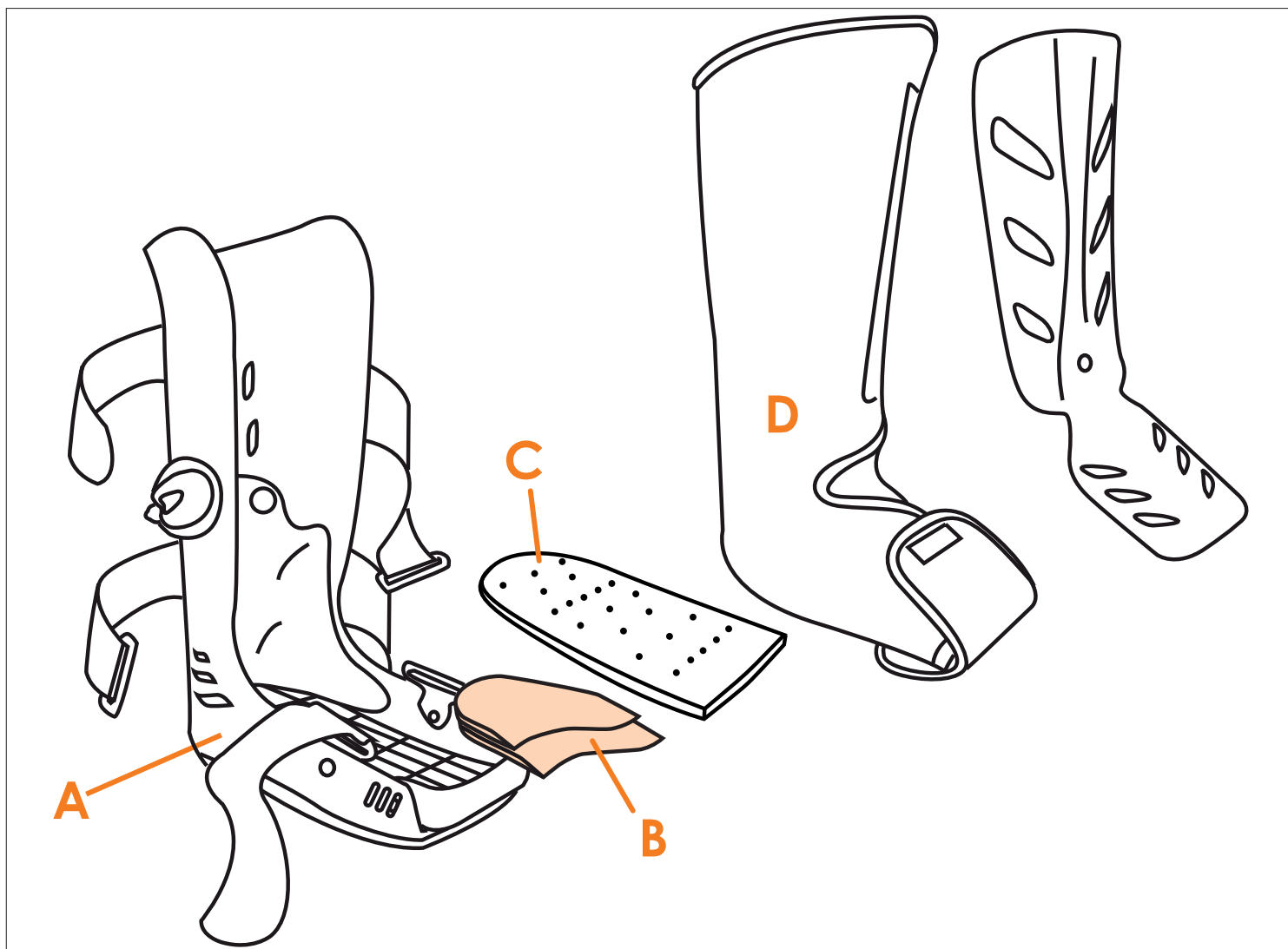
EN Place the wedges **B** under the sole **C** of Cirrus Airwalker.

1. Choose the correct size and number of wedges depending on to the patient's foot size and need for plantar flexion. Use the long wedges to fit larger foot size, the short wedges will fit smaller foot size.
2. Double-sided sticky tape is placed underneath each wedge **B**. Attach the wedges on top of each other using the sticky tape. Place the wedges **B** in the walker orthosis **A**.
3. Place the sole **C** over the wedges **B**, and then place the patient's foot with the padding **D** in the walker orthosis **A**.

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NordiCare Ortopedi och Rehab AB