Size	Hip Circumference	Height front	Height back	Art.no
X Small	65 - 80 cm	15 cm	20 cm	50200-00-30
Small	80 - 95 cm	15 cm	20 cm	50200-00-40
Medium	95 - 105 cm	15 cm	20 cm	50200-00-50
Large	105 - 115 cm	15 cm	20 cm	50200-00-60
X Large	115 - 125 cm	15 cm	20 cm	50200-00-70
2X Large	125 - 135 cm	15 cm	20 cm	50200-00-80

Application

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- EN** 1. Loosen Velcro straps.
 2. Put the spinal orthosis in place and fasten the Velcro at mid front over the belly/hip, without tightening too hard.
 3. Pull the straps firmly outward- forward to the desired stability. Close at front.

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- EN** Tuff is intended to ease back pain, sciatica and other pain on the lower back. It is also ideal for providing support for the pelvis, to ease residual SI problems after pregnancy, for example. In such cases, position the product farther down over the hips.

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- EN** To determine the size, measure hip circumference in cm. If the brace is to be used at waist height, use the waist measurement rather than the hip measurement.

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EN Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage. In case of pain, chafing or colour changes in the skin, remove the orthosis and contact the fitter.

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Material Composition

Polyester 50%, Nylon 30%, Spandex 20%. Latex free.

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EN Machine wash at 40°C (104°F). Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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For more information please consult our product catalogue or visit www.nordicare.se. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20/ DK 7070-1907, info@nordicare.se.