## INSTRUCTIONS FOR USE

# Atle™ – SPINAL ORTHOSIS

art no 50100, 50101

### Application

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**EN** 1. Bend the splints to spine shape and put in pockets.

2. Loosen Velcro straps.

3. Put the spinal orthosis in place and fasten the Velcro at mid front over the belly, without tightening too hard.

4. Pull the straps firmly outward- forward.

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**EN** To be used during lumbago, sciatica and other usually occurred back problems.

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### Heights and Measurement

Atle is available in two different heights in beige and black, see images on the next page. To determine the size, measure hip circumference in cm.

Model	Art.no	Height front	Height back
Atle Hög, beige	50101-08	24 cm	34 cm
Atle Hög, black	50101-00	24 cm	34 cm
Atle Låg, beige	50100-08	18 cm	27 cm
Atle Låg, black	50100-00	18 cm	27 cm

Size	Hip Circumference
Small	80 - 95 cm
Medium	95 - 105 cm
Large	105 - 115 cm
X Large	115 - 125 cm
XX Large	125 - 135 cm

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage.

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### Material Composition

Polyester 67%, Polyamide 14%, Rubber 12%, Elastane 7%.

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**EN** Machine wash at 40°C (104°F). Remove the splints. Seal the Velcros. Use washing bag. Do not bleach. Do not dry clean. Do not iron. Do not tumble dry.

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Please consult our product catalogue for more information. For further questions, contact your doctor, health practitioner, medical equivalent or NordiCare: +46 42-35 22 20, info@nordicare.se.