## INSTRUCTIONS FOR USE

# 50207 LYFT PLUS - ABDOMINAL BINDER

### Application

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**EN 1.** Position the girdle and secure the Velcro fastenings at the sides. If you have a large or hanging stomach/hernia, it may be preferable to put the girdle on while lying down. This will help the stomach/hernia to settle in the right place and ensure that the girdle provides optimal support.

**2.** Adjust until comfortable. Make sure it is not too tight.

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**EN** To be used post-operatively after abdominal surgery, to provide relief for surgical wounds and to prevent incisional hernia, also for pain relief and early mobilization of patients. Can also be used as an ostomy or hernia support.

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**EN** Prescription and treatment recommendations are to be made by a qualified medical practitioner. This also applies to initial fitting. A qualified medical practitioner should also supervise when using this product in combination with other medical products. Follow instruction for best result. Periodically check the condition of the Velcro to ensure they are free from grease and damage.

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### Heights and Measurement

Lyft Plus is 28 cm high at front and 23 cm high at the back. For correct size measure the hip circumference in cm.

Size	Hip Circumference
Small	80 - 95 cm
Medium	90 - 105 cm
Large	100 - 115 cm
X Large	110 - 125 cm
2X Large	120 - 135 cm
3X Large	130 - 145 cm
4X Large	140 - 155 cm

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**EN** Machine wash at 40°C (104°F). Can be washed up to 60°C (140°F) (high temperatures will reduce the life time of the Velcro). Seal the Velcros. Use washing bag. Do not bleach. Do not dryclean. Do not iron. Do not Tumble Dry.

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  
**NordiCare**

This product meets the requirements of the Medical Devices Directive 93/42/EEC. Class 1 medical device, non sterile.

NordiCare Ortopedi & Rehab AB  
Solrosvägen 1, SE-263 62 Viken, Sweden  
Tel +46 (0)42 35 22 20, Fax +46 42 35 22 21  
www.nordicare.se, info@nordicare.se