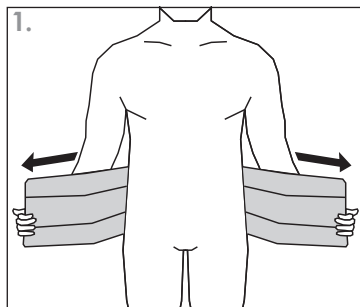
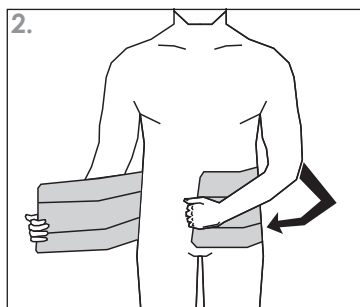
# HOW TO PUT IT ON

# AbdoCare™

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**EN** **AbdoCare** is an medical-surgical elastic belt that provides convenient and efficient support/compression of the torso. The binder is easy to fit, both immediately after surgery as well as by the patient him/herself. The size/degree of compression can easily be adjusted as the Velcro can be secured across the entire surface.


**AbdoCare** is available in two different heights and sizes, with large intervals. One size can be used as a universal size which therefore simplifies inventory management.

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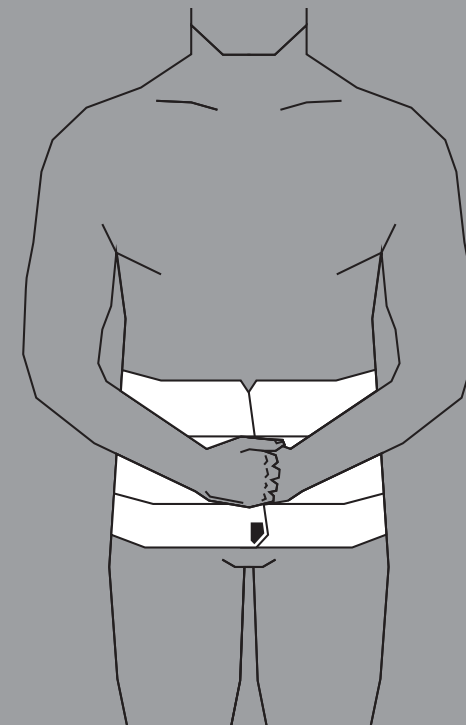
This product meets the requirements of the Medical Devices Directive 93/42/EEC. Class 1 medical device, non sterile.

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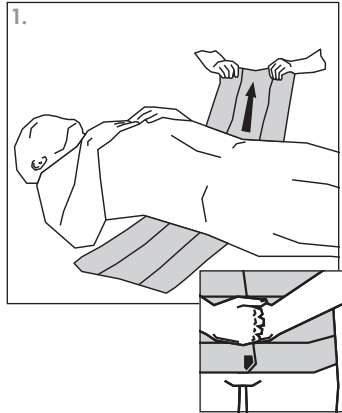
**EN** Position the girdle under the patient, the label should be located in the middle of the lower hem.

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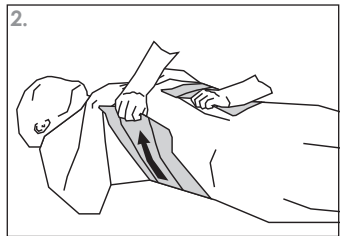
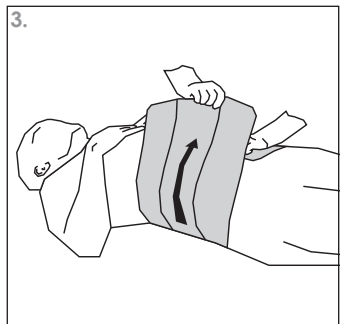
**EN** Stretch the other end until the required compression is obtained.

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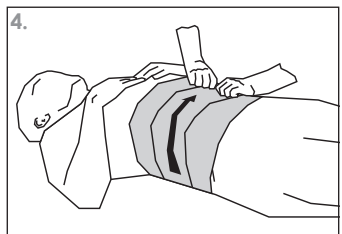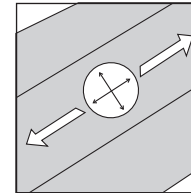
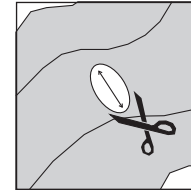
**EN** Position the girdle so that there is only one layer of fabric where the hole will be cut. Cut an oval hole, when the belt is stretched out the hole will become round.

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**EN** In order to determine the correct size, measure circumference of the abdomen of the intended placement. The size/degree of compression can easily be adjusted as the Velcro can be secured across the entire surface.

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**EN** AbdoCare can be washed at 95 °C approx. 10 times. The longevity of the product increases when washed at 40-60 °C. Do not bleach. Do not tumble dry. Flat dry. Close all Velcro before washing. Use a washing net.

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**EN** For post-operative care after thoracic and/or abdominal surgery to relieve the pressure to the wounds, reduce the risk of seroma/ infections/herniation, and prevent tearing of scars as well as early mobilisation of the patient. Recommended to be used for different types of minor hernias as well as while the ostomy belt is being manufactured.

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## Material

65% polyester, 19% nylon, 16% spandex. LATEX FREE